


## Connecting Ability Drive on a Permobil chair with an IOM


1. Start by removing the panel on the back of the chair that covers the RNet bus.



2. Remove the waterproof plug on the RNet bus bar.

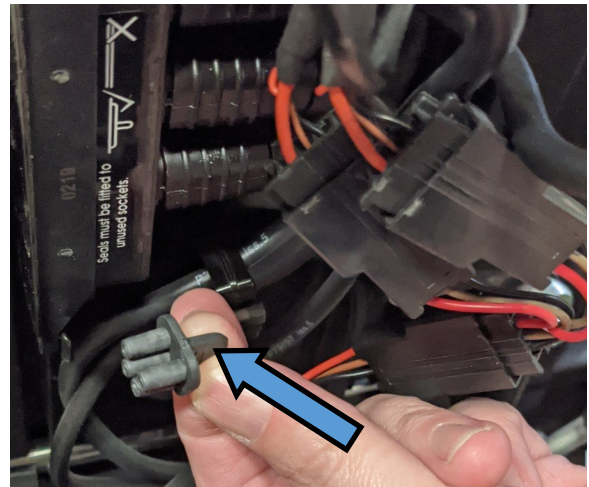

3. Plug the RNet cable into the RNet bus on the back of the chair. Replace the back panel.

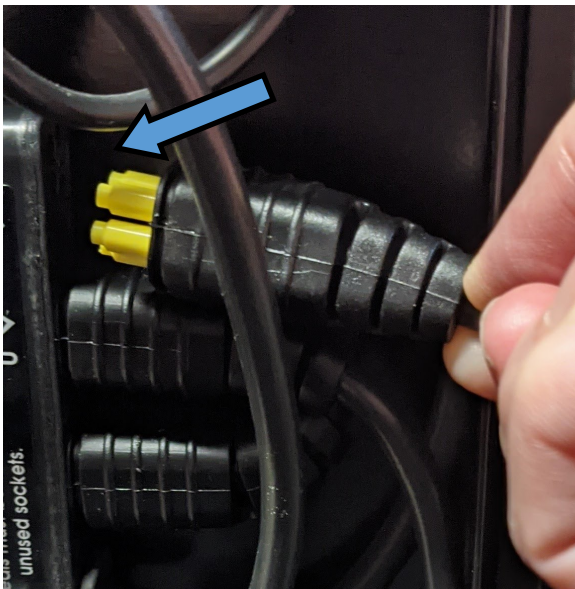

4. Plug the other end of the RNet cable into the IOM.



# Ability Drive®

5. Plug the 9Pin cable into the IOM at the INPUT Jack 1 port.  
Screw in the Cable.



6. Plug the Buddy Button into the IOM.


A Buddy Button should always be plugged in for safety.



7. Plug the other end of the 9Pin cable into the ADT.  
Screw in the 9Pin cable.



8. Plug the single screw end of the USBC cable into the ADT. Screw in the cable.

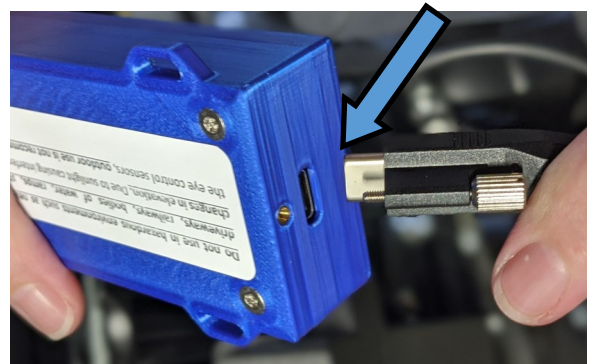

9. Plug the USBC cable into a supported device.

See the Tolt Technologies blog for a list of supported devices  
<https://toltechnologies.com/blog/what-devices-does-ability-drive-work-on/>

