

# Ability Drive®

## Connecting Ability Drive on a Permobil chair with an Omni 2

1. Plug the 9Pin cable into the Omni 2 at the input **PORT 1**.  
Screw the cable in.




2. Plug the other end of the 9Pin cable into the ADT.  
Screw in the 9Pin cable.



3. Plug the Buddy Button into the IOM.  
A Buddy Button should always be plugged in for safety.



4. Plug the single screw USB-C cable into the ADT.  
Screw in the cable.

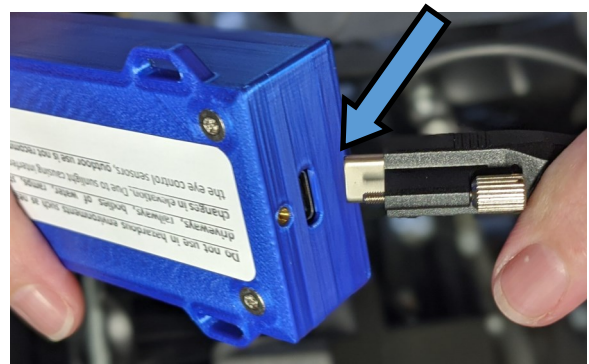

6. Plug the USB-C cable into a supported device  
See the Tolt Technologies blog for a list of supported devices

<https://toltechnologies.com/blog/what-devices-does-ability-drive-work-on/>

